

# LFA REUSABLE MEDICAL FACE SHIELD

CODE 664-182



## SPECIFICATION SHEET

- PETG Material  
(Polyethylene Terephthalate Glycol)
- Foamless Design - Reusable
- Can be used together with an  
N95 mask and glasses if required



 1800 681 544

 [sales@lfafirstresponse.com.au](mailto:sales@lfafirstresponse.com.au)

 [www.lfafirstresponse.com.au](http://www.lfafirstresponse.com.au)

# LFA REUSABLE MEDICAL FACE SHIELD

CODE 664-182

## OVERVIEW

LFA has developed this face shield for the COVID-19 pandemic. The core benefit of the reusable shield is to conserve raw materials and get quality shields to the front line.

- TGA: ARTG 333131
- PETG Material (Polyethylene Terephthalate Glycol).
- Foamless Design – Reusable.
- Ships Flat Packed (Easy user assembly).
- Superior Chemical Resistance.
- Full Face Protection.
- One Size Fits All.
- Secure Fit
- Can be used together with an N95 mask and glasses if required

## CLEANING GUIDE - LFA REUSABLE MEDICAL FACE SHIELD

After first use, the outside of protective eyewear/face shields may be contaminated.

- Step 1:** Remove eyewear/face shield by tilting the head forward and lifting the head band or ear pieces. Avoid touching the front surface of the eyewear/face shield.
- Step 2:** Obtain Clinell Universal wipes, Quaternary Ammonium Compound or Sodium Hypochlorite Solution 10,000ppm (Or equivalent approved covid-19 sanitisers).
- Step 3:** Using the wipe/solution, start with the clear shield first then work through to the visor and straps, ensuring full coverage of face shield.
- Step 4:** Using Put shield aside and leave until dry before use. This is a cleaning guide that should be completed between shifts by the user, to ensure it is sterile and ready for the next day

*Life Happens...  
Be Prepared*

**LFA**  
*REUSABLE  
MEDICAL  
FACE SHIELD*

**CODE 664-182**

**PRODUCT IMAGES**



*Life Happens...  
Be Prepared*